**SANOSKIN OXY is a Medical Device** **MD**

## **NEDERLANDS**

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### **ENGLISH**

**SanoSkin®-OXY** (ointment made of vegetable oil)

15 - 30 - 50 gram tubes

### **Product description**

SanoSkin®-OXY is a fatty ointment that consists of ozonated vegetable oil (100% olive oil). When the ointment comes into contact with wound fluid/water, ozonides gets released. Those ozonides create an acid and oxidative environment that is unfavourable for micro-organisms like bacteria, yeast and fungi.

### **Indications**

SanoSkin®-OXY is used in the treatment of superficial wounds. The gel creates a moist wound healing environment that helps the autolytic debridement of necrotic tissue. Do not use in eyes or mucous layers.

### **Contra-indications**

Patients who are known to be sensitive to the ointment or one of its components.

### **Precautions**

The presence of bacteria in wounds is a normal phenomenon and is no contra indication. If an infection is suspected or is developing during the use then a correct antibiotic therapy should be started. Depending on the underlying pathology the wound care treatment should always be accompanied by the correct medical treatment and supervision.

SanoSkin®-OXY creates a moist wound climate which stimulates the debridement of necrotic tissue. Due to this it is normal that the wound surface becomes bigger.

### **Pregnancy and breastfeeding**

There is no information available about the use of SanoSkin Oxy during pregnancy and breastfeeding. Are you pregnant or Breastfeeding? Please contact your doctor or pharmacist.

### **Application**

Use SanoSkin®-OXY always exactly as your doctor or pharmacist has told you, or how it is explained in these insert.

1. Clean the wound.
2. Open tube and remove seal in an antiseptic way.
3. Bring ointment directly on/in the wound and make sure the ointment does not come above the wound edges.
4. Cover the ointment/wound with a moisture retention dressing.

### **Remove**

SanoSkin®-OXY should be changed when leakage of the secondary dressing occurs or when the secondary dressing needs to be removed. The ointment can be cleansed out of the wound with a sterile saline solution. Dressing change is depending on the wound fluid production of the wound, but is usual between 2 -7 days.

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