SANOSKIN OXY is a Medical Device **MD**

NEDERLANDS

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ENGLISH

SanoSkin®-OXY (ointment made of vegetable oil)
15 - 30 - 50 gram tubes

Product description

SanoSkin®-OXY is a greasy ointment consisting of ozonated vegetable oil. (100% olive oil)

When the ointment comes into contact with wound fluid/water, ozonides are released. These ozonides create an acidic and oxidative environment that is unfavourable for micro-organisms like bacteria, yeast and fungi.

Indications

SanoSkin®-OXY is indicated for the treatment of superficial wounds.

The ointment creates a moist oxidative wound healing environment which stimulates the process of autolytic debridement of necrotic tissue. Do not use on the eyes or mucous membranes.

Contra-indications

Patients with known hypersensitivity to the ointment or any of its components

Precautions

The presence of bacteria in wounds is a normal phenomenon and is not a contra indication. If an infection is suspected or is develops during use, then an appropriate antibiotic therapy should be initiated. Depending on the underlying pathology, wound care should always be accompanied by appropriate supportive medical treatment and supervision.

SanoSkin®-OXY creates a moist oxidative wound environment which promotes debridement and necrotic tissue removal.

Due to this the wound surface can become slightly larger at the beginning of the treatment.

Pregnancy and breastfeeding

There is no information available about the use of SanoSkin Oxy during pregnancy and breastfeeding. Are you pregnant or breastfeeding? Please ask your doctor or pharmacist.

Application

1. Clean the wound.
2. Open the tube aseptically.
3. Immediately apply the ointment to / in the wound and make sure the ointment does not come above the wound edges.
4. Cover the ointment/wound with a moisture retention dressing.

Remove

SanoSkin®-OXY should be changed when leakage of the secondary dressing occurs or when the secondary dressing needs to be removed. The ointment can be cleansed out of the wound with a sterile saline solution. Dressing change depends on the wound fluid production of the wound, but is on average every 2 to 7 days.

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© SanoMed Manufacturing BV 2020 Barring Spelling errors, SanoMed Manufacturing BV shall at all times retain the right to change its products and assortment. Doc: OXY 30 I02020v5