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20 - 30 - 50 gram tubes

Product description

SanoSkin Melladerm® PLUS contains: honey, glycerine, propylene glycol and PEG 4000. When the gel is in contact with the wound, fluid is extracted from the surrounding tissues. The osmotic action creates a moist wound-healing environment. Honey together with the aforementioned moist wound healing environment stimulates the wound healing process, facilitates autolysis and promotes better epithelial cell migration. The gel will dilute gradually and can be removed easily with a wound cleanser. *SanoSkin Melladerm® PLUS* is a primary wound dressing that can be covered by most commonly used secondary dressings.

Indications and Intended use

Superficial Chronic wounds - Decubitus, Ulcus Cruris, diabetic foot wounds, Fungating wounds (deodorising and debriding).

Contaminated Acute wounds – surgical wounds (postoperative wounds), Traumatic wounds (superficial wounds, cuts), Small burn and laser wounds (1° and 2° degree). Melladerm Plus can be uses both by professionals and laymen.

Contraindications

Do not use if package is damaged or opened. No contraindications have been reported to date. Do not use the gel if the patient is highly prone to infection e.g. a patient with very large burn wounds. Do not use on patients with a known hypersensitivity to the gel or its components.

Instructions for use

1. Remove lid and open tube in an aseptic way.
2. Squeeze the tube to extract the gel and apply it. without touching the wound.
3. Use a (sterile) spatula when necessary.
4. Place a dressing on top of the gel to cover the wound.
5. *SanoSkin Melladerm® PLUS* can remain on the wound up to 48 hours on average. Reapply the gel on the wound as soon as the gel has disappeared.

Pregnancy, breastfeeding and use in Paediatric patients (0-15 years)

There is no information available about the use of *Melladerm Plus* during pregnancy and breastfeeding. Are you pregnant or Breastfeeding? Please contact your doctor or pharmacist. No contraindications are known to date for the use of *Melladerm Plus* in Paediatric patients.

Precautions and observations

- * If necessary consult a healthcare professional for the appropriate medical treatment.
- * Inspect and clean the wound with a wound cleanser.
- * Bacterial colonisation of chronic wounds is always present and is not a contraindication for using *SanoSkin Melladerm® PLUS*.
- * Due to the autolytic debridement the wound may appear deeper after the first dressing changes.
- * Seek medical supervision if signs of infection occur.
- * *SanoSkin Melladerm® PLUS* is for single patient use only.
- * When the condition of the wound deteriorates, treatment with *SanoSkin Melladerm® PLUS* should be stopped.
- * After the packaging has been opened, the storage life of the product is equal to the expiry date printed on the packaging as long as the instructions for use have been respected.
- * Do not use the product continuously for more than 30 days.

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© SanoMed Manufacturing BV 2020 Barring Spelling errors,